

3-LAYER MEDICAL MASK m-c24

Three-layer masks, which are medical devices, meet the requirements of EN 14683: 2019 + AC2019. They are made of polypropylene non-woven fabric, an additional filtering layer, a stiffening element and two fastening elastics.

Instructions for m-c24 medical face masks use:

1. Take the mask out of the packaging.
2. After taking the mask out of its packaging, do not put it back in a place that could contaminate the mask.
3. Determine which side of the mask is the top. The side of the mask that has a stiff bendable edge is the top and is meant to mold to the shape of your nose. Check which side of the mask is the front. The colored side of the mask is usually the front and should face away from you, while the white side touches your face. The elastics are placed behind the ears. Place a loop around each ear.
4. After putting the mask on the face, pull down the bottom of your mask, so that it covers your mouth and chin. Press the flexible upper edge of the mask along the bridge of your nose. There should be no gaps around the nose or under the chin.
5. Masks cannot be worn under the nose or hung over the neck.

Keep in mind that:

The filtration efficiency and protective ability of the mask diminishes as the mask becomes wet, torn or displaced. The higher the protection value of the mask, the longer it will maintain filtration, however there is no set rule regarding the length of time when it can be used as it depends on the level of humidity, respiration rate, nasal discharge, speaking etc.

The mask should be replaced at each patient admission or appointment.

If the mask becomes wet, dirty or damaged, replace it immediately.

How to store the mask:

The product should be stored in clean and dry conditions at a temperature between - 5 ° C and + 35 ° C, with a maximum relative humidity of <80%. When storing or transporting the product, use its original packaging.

Contaminated products should be disposed of as hazardous waste in accordance with national regulations.

The shelf life is 30 months from the production date.


Manufacturer:

WKM

Ul. Brodowicza 11/2, 31-518 Kraków Tel. +48 12 378 32 77 Mobile. +48 531 944 542

Fax. +48 12 376 46 31

E-mail: bok@wkm-medical.com.pl

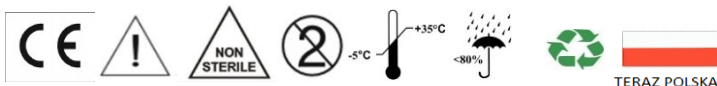

3-LAYER MEDICAL MASK m-c24 FILTER IIR, type IIR

50 pcs

See the packaging

See the packaging

30 months from the production date

Filtration efficiency of BFE bacteria > 98%

A stiffening element on the nose

3-ply

Read the user instruction before use

